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ZeoBentMED VULKANMINERALIEN

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ZeoBentMED VULKANMINERALIEN

BRIEF DESCRIPTION

Detox Powder or Detox Powder Ultra Fine or Detox Capsules
Natural medical device to support the binding and elimination of heavy metals from the organism.
100% NATURE

COMPOSITION

80% clinoptilolite zeolite + 20% montmorillonite bentonite or 50% clinoptilolite zeolite ultrafine + 50% montmorillonite bentonite, 50% clinoptilolite zeolite ultrafine + 50% montmorillonite bentonite in 500 mg cellulose capsules.

PURPOSE

This natural medical device supports the binding and elimination of heavy metals (Pb, Hg, Cr, Cd, Al and Ni) from the organism of adult persons. It may additionally reduce symptoms associated with chronically elevated heavy metal levels, such as fatigue and headache.

WARNING AND SAFETY INSTRUCTIONS

Interrupt the application if you notice unexpected side effects and consult your doctor. If constipation occurs, increase fluid intake, reduce dosage and consult your doctor if necessary. The medical device may only be used after medical consultation in patients with existing constipation, impaired kidney function, existing disease, during drug therapy or chemotherapy or radiotherapy. Breastfeeding or pregnant women, children and adolescents (under 18 years) must contact their doctor before using the medical device. Do not inhale product. Do not use at the same time as alcoholic, caffeinated and acidic drinks such as grapefruit, orange, lemon and pineapple juice. In case of eye contact, rinse the eyes thoroughly. Ensure sufficient fluid intake in the form of water or other non-alcoholic drinks during use. Not intended for continuous long-term use. Stop using if you notice unusual effects.

Side effects: Constipation (obstipation) may occur, especially during the first week of use. In very rare cases, flatulence, stomach pain, nausea, dizziness and tiredness may occur at the beginning of intake.

Interactions: If you are taking medication at the same time, make sure that there is a time gap of 2 hours. Take medicines before taking Zeolite Clinoptilolite unless otherwise prescribed by your doctor.

MODE AND DURATION OF USE OF ZEOLITE ULTRAFAINE + BENTONITE POWDER

Medical device for oral use: Stir ½ teaspoon (= 1 g) into a glass of water (250 ml) 2-3 times daily and drink immediately with meals. The maximum duration of intake is 12 weeks. In case of repeated use, a break of at least one week should be observed.

MODE AND DURATION OF USE OF ZEOLITE + BENTONITE POWDER

Medical device for oral use: Stir 1 teaspoon (= 2 g) into a glass of water (250 ml) 2-3 times daily and drink immediately with meals. The maximum duration of intake is 12 weeks. In case of repeated use, a break of at least one week should be observed.

MODE AND DURATION OF USE OF ZEOLITE + BENTONITE (CAPSULES)

Medical device for oral use: Take 2 capsules 2-3 times daily with a glass of water (250 ml) with meals. The maximum duration of intake is 12 weeks. In case of repeated use, a break of at least one week should be observed.

STORAGE AND USABILITY

Keep out of reach of children. Close the tin tightly after use. Store the product in a dry place at room temperature. Protect from direct sunlight.
Do not use the product after the expiry date. Use by: see expiry date on tin

DISPOSAL

Contents and packaging can be disposed of with household waste. Please place violet glasses in the green glass container.

NOTES ON INCIDENTS

All serious incidents related to the medical device shall be reported by the user to the manufacturer and to the competent authority of the Member State in which the user is established.

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Natur Vital Handels GmbH
Johner Medical Schweiz GmbH